

## INSTALLATION INSTRUCTIONS

( pinch emptier, manual chair )

*Please Read Before Starting !!!*

It has been our experience that each installation is in some way unique. Please consider this set of instructions to be a general guide, some deviation will be necessary to meet your individual situation.

### **This Kit Contains**

- 1 Main (seat) Harness
- 1 Leg Harness
- 1 Switch (1 or 2 button)
- 1 Adapter (power converter)
- 1 Lithium Ion Power Bank (5V, 6000mah, 2.4V)
- 1 Micro USB cable
- 1 Power Adapter Bag
- 1 Pinch Emptier
- 7 Plastic cable ties

Please check that all of these are present.

### **MAIN HARNESS**

1. -This harness has 3 connectors, 1 of which has a jumper inserted and is sealed. Do not remove this jumper. The harness mounts to the chair frame, under the seat on the same side as the emptier.  
-The correct placement will have the single connector at the front of the chair where it mates with the leg harness. Secure the wires to the frame using the plastic ties. Take care to avoid pinching the wires between chair parts. Allow enough slack if the chair will be folded.

**SWITCH** The switch is usually mounted on the arm rest. However it can be mounted virtually anywhere the user desires. Its connector is COLOUR CODED IS **BLUE**

1. ( 2 Button ) -Find the most comfortable position for the user.  
-Apply the Velcro strip to the switch box.  
-Remove the protective sheet from the second side and press the box into place.  
-Depending on the location on the chair, it may be necessary to add shims or otherwise add stability to the switch box.
2. -Secure the switch box harness to the chair frame with plastic ties, remembering to allow enough slack if the chair will be folded.
3. ( 1 Button ) -Find the most comfortable position for the user.  
-Clean this location with a mild detergent solution and dry thoroughly.  
-Remove the plastic backing from the 2 sided foam adhesive patch and press the switch firmly in place.

### **Power Supply:**

1. -The power supply is a Lithium Ion Power Bank which is charged using any USB connection such as your computer or iPad charger. When it eventually wears out, you can replace it with a locally acquired equivalent. .  
  
-PLEASE SEE THE POWER BANK INSTRUCTION AND WARNING SHEET BEFORE USING THE SYSTEM.
2. -The bag supplied with this kit should be mounted under the chair using both adjustable straps The position chosen should allow the chair to be folded without damaging the bag and its contents. If necessary the bag is easily removed. Place the adapter, USB cable and the Power Bank into the bag.  
- Feed the power harness (COLOUR CODED **YELLOW**) out of the bag, feed the switch harness into the bag and connect all of the colour coded connectors (**BLUE, GREEN**). Plug the USB cable into the Power Bank (not colour coded). Plug the power harness into the seat harness. close the Velcro flap  
  
.
4. - Connect the leg harness to the seat harness and then connect the pinch emptier. Test that the system is working. There may be a slight hesitation when it is first activated. Just push the button a few times if this occurs.

### **EMPTIER**

In normal usage, emptier should be held in place by the leg support band.

The leg bag outlet hose should be cut leaving enough hose for one of the 2 ended barbed fitting to be inserted, the pinch emptier hose is connected to the other barbed end. Any excess slack in the hose can be adjusted by sliding the pinch emptier hose downward. Position the lower portion of the hose so that it is secured and concealed by the foot support heel strap.