

INSTALLATION INSTRUCTIONS

(Mini emptier, manual chair)

Please Read Before Starting !!!

It has been our experience that each installation is in some way unique. Please consider this set of instructions to be a general guide, some deviation will be necessary to meet your individual situation.

This Kit Contains

- 1 Main (seat) Harness
- 1 Leg Harness
- 1 Switch (1 or 2 button)
- 1 Adapter (power converter 5V to !2V)
- 1 Lithium Ion Power Bank (5V, 6000mah, 2.4V)
- 1 Micro USB cable
- 1 Under Chair Storage Bag
- 1 Mini Emptier
- 7 Plastic cable ties

Optional: Emptier Support Band

Please check that all of these are present.

MAIN HARNESS

1. -This harness has 2 connectors, 1 of which colour coded Yellow The other White. This harness mounts to the chair frame, under the seat on the same side as the emptier. The Yellow coded cable from the Adapter plugs into the Yellow connector.

-The White connector at the front of the chair where it mates with the leg harness. Secure the wires to the frame using the plastic ties. Take care to avoid pinching the wires between chair parts. Allow enough slack if the chair will be folded.

SWITCH The switch is usually mounted on the arm rest. However it can be mounted virtually anywhere the user desires. Its connector is colour coded **BLUE** and plugs into the corresponding blue connector from the adapter.

1. (2 Button) -Find the most comfortable position for the user.
 - Apply the Velcro strip to the switch box.
 - Remove the protective sheet from the second side and press the box into place.
 - Depending on the location on the chair, it may be necessary to add shims or otherwise add stability to the switch box.
2. -Secure the switch box harness to the chair frame with plastic ties, remembering to allow enough slack if the chair will be folded.
3. (1 Button) -Find the most comfortable position for the user.
 - Clean this location with a mild detergent solution and dry thoroughly.
 - Remove the protective backing from the 2 sided foam adhesive patch and press the switch firmly in place.

Power Supply:

1. -The power supply is a Lithium Ion Power Bank which is charged using any USB connection such as your computer or iPad charger. When it eventually wears out, you can replace it with a locally acquired equivalent. .

-PLEASE SEE THE POWER BANK INSTRUCTION AND WARNING SHEET BEFORE USING THE SYSTEM.

2. -The bag supplied with this kit should be mounted under the side or back chair using both velcro straps. The position chosen should allow the chair to be folded without damaging the bag and its contents. If necessary the bag is easily removed. Place the adapter, USB cable and the Power Bank into the bag.
- Feed the power harness (COLOUR CODED **YELLOW**) out of the bag, feed the switch harness into the bag and connect all of the colour coded connectors (**BLUE, GREEN**). Plug the USB cable into the Power Bank (not colour coded). Plug the power harness into the seat harness. close the Velcro flap.
4. - Connect the leg harness to the seat harness and then connect the Mini emptier. Test that the system is working. There may be a slight (fraction of a second) delay when it is activated. Just hold the button down.

EMPTIER

In normal usage, emptier should be held in place on the foot rest, unless the optional emptier support band has been purchased.

The leg bag outlet hose should be cut leaving enough hose length to attach to the inlet barbed fitting on the emptier. The off cut piece of hose can be fitted to the outlet barbed fitting if an emptier support band is being used.