

## CARING FOR YOUR EMPTIER

( pinch emptier )

Your new emptier is very reliable and requires little maintenance. Clean water is what we recommend for a daily flushing of the tubing. You may use a germicide or vinegar as long as you flush with water last. Dry the tubing before reinserting it into the pinch mechanism. **Never allow any** liquid to enter the pinch mechanism.

The following should be incorporated into your end of day routine.

**VERY IMPORTANT!!**

**FOLLOW TO THE LETTER OR YOUR WARRANTY WILL BE VOID!!!!**

### Shut Down Procedures

1. Empty the leg bag using the emptier.
2. Close the clamp or valve on the leg bag.
3. Remove the tubing from the emptier. (slip it out of the pinch mechanism)
4. Unplug the leg harness from the emptier and remove the emptier plus support band from the user's leg.
5. Unplug the emptier tubing from the leg bag tubing.
6. Wash the emptier tubing in the bathroom sink with fresh water. Dry & reinsert the washed tubing into the emptiers pinch mechanism.

### Chair Servicing

- 1) Always have emptier installation instructions and wiring diagram handy when service personnel arrive.
- 2) Always test emptier **BEFORE** battery cover is replaced.
- 3) Always **INSIST THAT EMPTIER WORK AS IT DID BEFORE.**

## TROUBLE SHOOTING

1. -Check that all connectors are plugged in properly.  
-Check that all wires are intact (including their attachment to the connectors) and not broken or bare.
2. -Check that the battery connections are the correct voltage for your emptier.  
(12VDC)
3. -Check the fuse. (1A 250V fast blow)
4. -Check that the batteries are charged.
5. -Check that both switches are in the down position when you are attempting to operate the emptier.
6. -If the fuse blows for an unknown reason have the system checked by a qualified electrical technician before operating again.
7. -Contact:  
Assistive Designs  
P.O. Box 20015  
55 Muskoka Rd. 118 West  
Bracebridge, Ontario  
Canada, P1L 2H6  
fax: (705) 646-8845  
e-mail: [emptier@assistivedesigns.com](mailto:emptier@assistivedesigns.com)  
[www.assistivedesigns.com](http://www.assistivedesigns.com)

WARRANTY - Assistive Designs warrants each of its products to be free from defects in material and/or workmanship for a period of twelve (12) months from the date of purchase by the customer. This warranty is expressly in lieu of all other warranties, expressed or implied. This warranty shall not apply to Assistive Designs equipment which shall have been repaired or altered by unauthorized service personnel, or which has been subjected to misuse, damage, or operated in a way other than its intended use. This warranty does not cover damage in transit. ASSISTIVE DESIGNS, ITS EMPLOYEES AND OFFICERS SHALL NOT BE LIABLE FOR ANY SPECIAL, CONSEQUENTIAL OR INCIDENTAL DAMAGES, OR RESULTANT COSTS HOWEVER CAUSED.